smart

Quick Reference
Kurzanleitung
Guía rápida

EURO Lyser 
DIAGNOSTICA

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
Switching the Eurolyser smart laboratory photometer on and off

Start up the smart laboratory photometer by connecting the power supply (AC adaptor plug) to the device and plugging the power cable into the wall socket.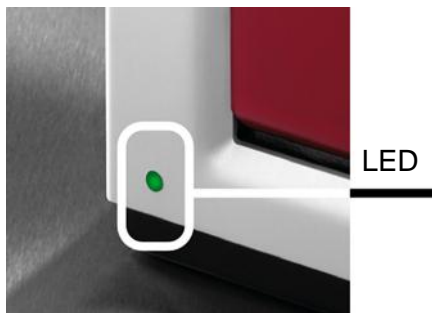


It is not necessary to disconnect the smart laboratory photometer from the power supply after analyses have finished.

The device is equipped with an **automatic standby function**. When it remains idle in the main menu for 10 minutes with no further operating commands, it switches into standby mode and dims the display lights.


The green LED light on the front remains illuminated.

To switch from standby mode to **operational mode**, just touch the display.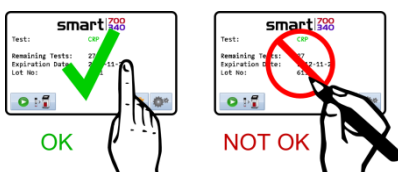

- Use only your fingers to touch the display! Never use a hard or pointed object.

- Do not use liquid cleaners on the display. Use a lint-free, slightly damp cloth only.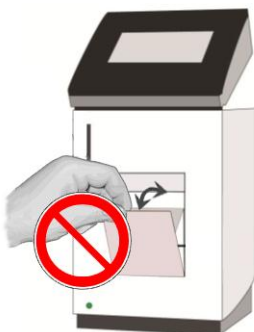

- Do not attempt to open or close the door manually.


Analyzing a patient's sample

1. Start-Up menu

Take the provided RFID card out of the testing kit package and insert it into the smart laboratory photometer.

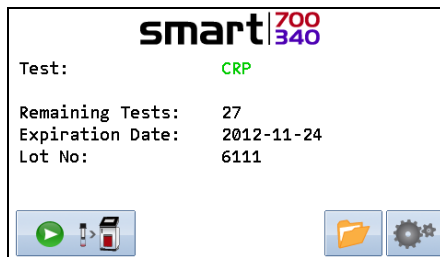
2. Main menu

The smart laboratory photometer automatically reads the card, displays the type of test, number of tests, the expiry date and lot number.

Touch  . The door opens automatically and the input menu appears.

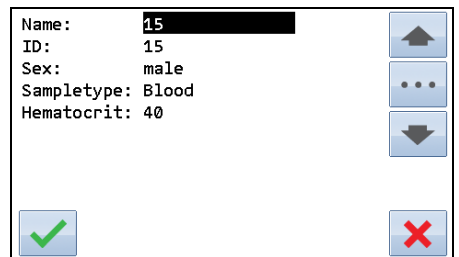3. Entry menu


Touch . The ABC menu appears. To navigate within the input menu, use  and .


To leave a menu or to cancel an entry, touch .

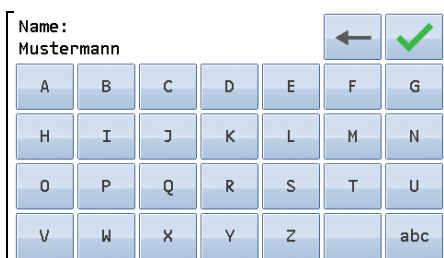4. ABC-Menu


Enter the name and confirm input with . Use  to delete an incorrect input.

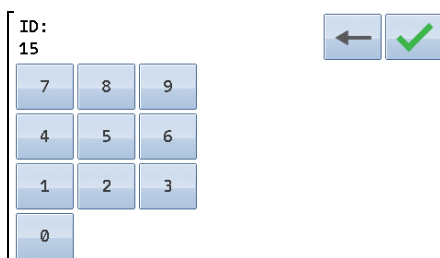5. Numeral menu


Enter the patient ID and confirm it with . Use  to delete an incorrect input.

6. Input Gender / Control Menu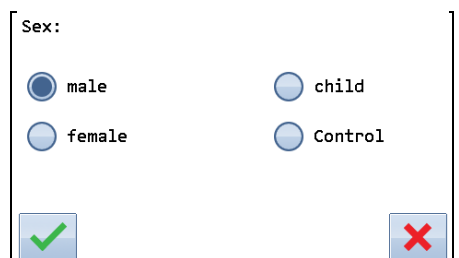


Touch **male**, **female**, **child** or **Control** and confirm with .

7. Sampletype menu
(not for all tests)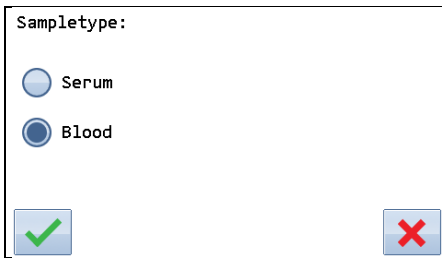


Touch Serum or Blood and confirm the selection with .

8. Hematocrit menu
(not for all tests and/or sampletypes)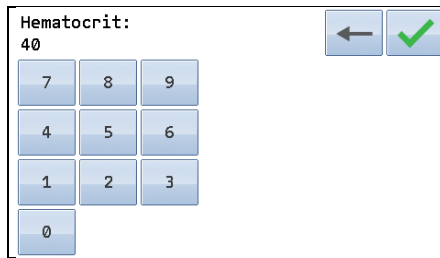

Enter the hematocrit value and confirm it with .

9. Start analysis menu

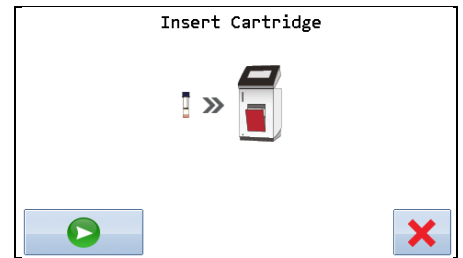Insert the test cartridge. Touch : The door closes and the smart laboratory photometer runs the test automatically.


- Be sure to handle the test cartridge according to the instructions on the package insert as well as make sure the test cartridge is firmly capped before you insert it into the smart laboratory photometer.
- Be sure the test cartridge is completely put into the proper opening in the smart laboratory photometer.
- Use only test cartridges that are approved by the manufacturer. Otherwise severe damage to the smart laboratory photometer and/or incorrect results can occur.
- Do not attempt to open or close the door manually.

Viewing and processing the test results

The Eurolyser laboratory photometer can store up to 250 patient and control results in its memory. Once capacity is exceeded, oldest results are deleted.

The following parameters are saved from each test:

1. Single result menu

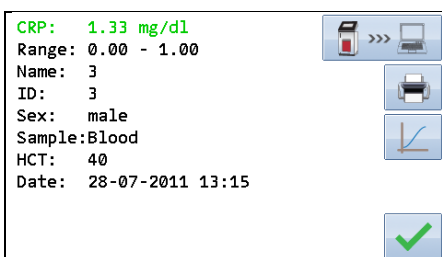
The result is shown. Touch to print it out. Touch to export it to an HIS/LIS or PC, or touch to display the photometric data. Touch to close the menu and end the test.

2. Remove cartridge menu

The smart laboratory photometer automatically opens the door. Remove the test cartridge and discard it. Touch to close the door. The main menu now appears. Touch to go to the overall result menu.

3. Overall result menu

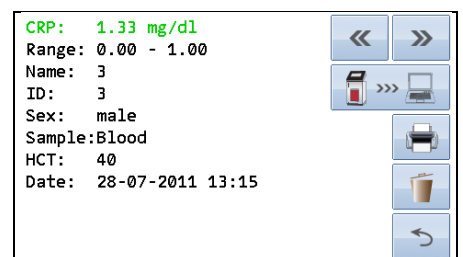Use and to scroll back and forth within the results. Touch to print the result(s). Touch to transmit the result(s) to a PC. Touch to delete the result(s). Touch to return to the main menu.

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Eurolyser Diagnostica GmbH