

CUBE Tablet PC

Starter Manual

for SN Tc#####




Eurolyser Diagnostica GmbH
Bayernstraße 11a
5020 Salzburg, AUSTRIA

Tel: +43 662 432100
Fax: +43 662 432100 50

TABLET PC STARTER MANUAL

The Tablet PC starter manual will guide through the start-up process and instructs setting up time and date, changing the system language, connecting to Wi-Fi, using the Browser as well as updating the CUBE App. Please refer to the Eurolyser CUBE quick reference and/or to the user manual for further guidance of setting up a connection from the Tablet PC to an instrument and operating the instrument.


The Tablet PC Buttons and Interface

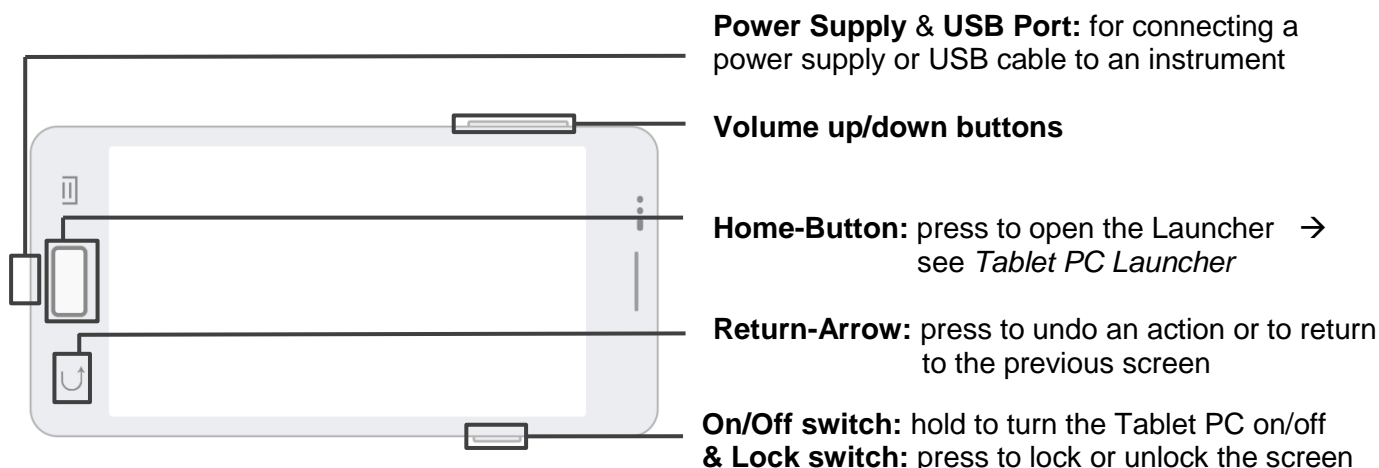

Starting and operating the Tablet PC

Press the On/Off switch to turn on the Tablet PC. After completing the Start-Up the Tablet PC Launcher appears.

Tablet PC Launcher

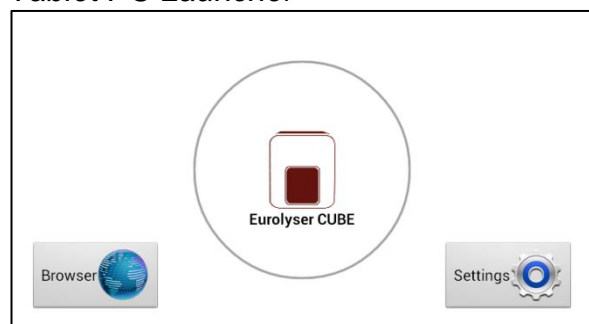


If the Tablet PC is not connected to the Instrument or by pressing the Home-Button the Tablet PC Launcher appears as well.

Tapping *Browser* opens the Android web browser

Tapping the *large circle* starts the Eurolyser CUBE application

Tapping *Settings* opens the Android settings menu


Setting the language of the Tablet PC

The language of the Tablet PC can be changed by pressing the Home-Button on the Tablet PC (the launcher appears), then pressing  to open the settings.

Tab on *Account* and select *Language & input* in the category “PERSONAL”. Tap *Language* and choose the desired language. Press the Home-Button to return to the Launcher.




Note: The CUBE app supports several languages. If a system language is chosen that is not supported by the application, English will be used instead.


Setting date and time of the Tablet PC

The date and time of the Tablet PC can be changed by pressing the Home-Button on the Tablet PC (the launcher appears), then pressing  to open the settings. Swipe left and select *Date & time* in the category “MORE”. Tap *Set date* and choose the current date. Tap *Select time zone* and choose the current time zone. Tap *Set time* and set the current time. Press the Home-Button to return to the Launcher.


Setting up a Bluetooth connection

Follow the steps below to establish a Bluetooth connection:


- 1) To set up a Bluetooth connection, press the Home-Button on the Tablet PC (the Launcher appears), then pressing  to open the settings.
- 2) Turn the  switch at *Bluetooth* to the **RIGHT SIDE**. The switch is now **ON** () and will appear green.
- 3) Tap *Bluetooth* to go to the Bluetooth settings. A list appears that lists all pair-able devices in the current location. Check the serial number on the back-side of the instrument and select the correct device from the list.
- 4) Another dialog appears and a PIN is needed for pairing. Type in “1234” and press **OK** to pair instruments to the tablet.

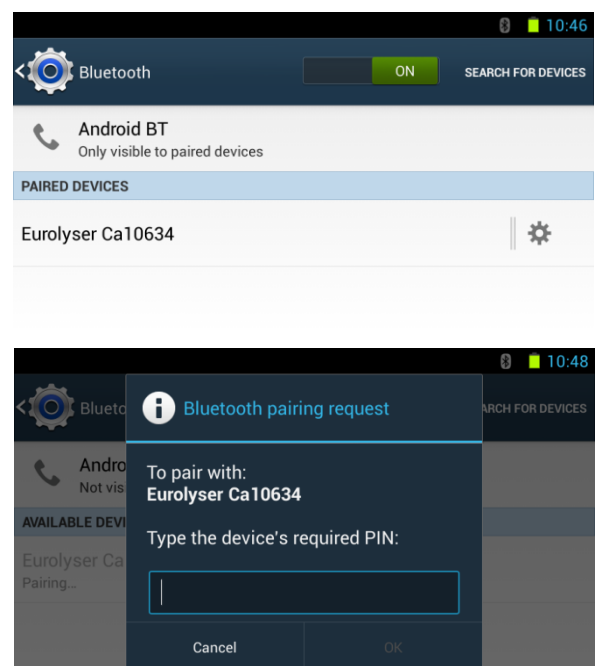

- 5) Press the Home-Button on the Tablet PC. The launcher appears and the application can be started by pressing the application icon.
- 6) A dialog appears which shows the paired device. A connection can be established by tapping on its name.
- 7) After successfully connecting to the device the application shows now the paired devices' details and the Bluetooth connection is now established.

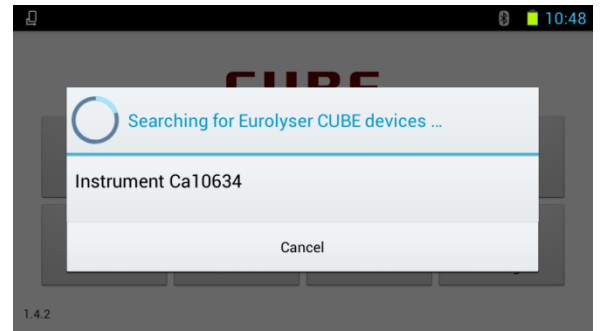


Set up a wireless network connection

To set up a wireless network connection, press the Home-Button on the Tablet PC (the Launcher appears), then press  to open the settings. Turn the switch at *Wi-Fi* to **ON** and tap *Wi-Fi* in the category "WIRELESS & NETWORKS". The Wi-Fi network list will show up.

Follow the steps below to establish a Wi-Fi network connection:


Step 1:

Wi-Fi Network List

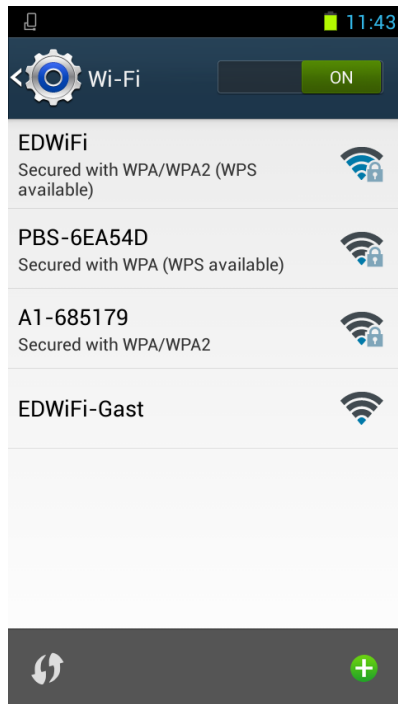


Step 2:

Wi-Fi Network Connection Mask

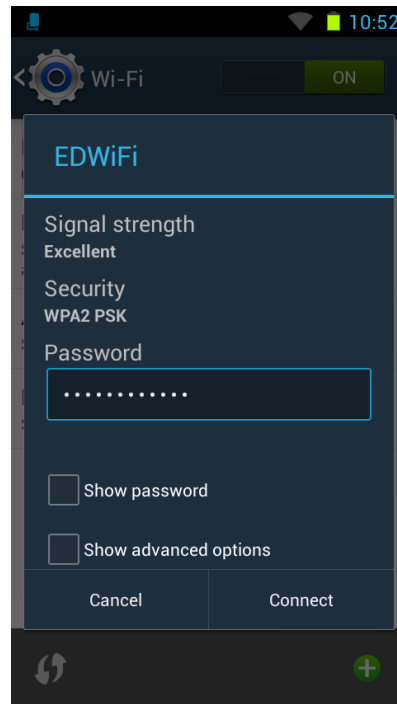

Step 3:

Wi-Fi Network List

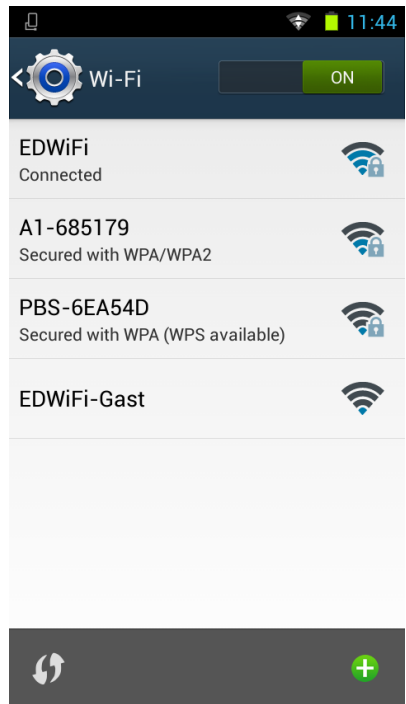

The Wi-Fi network list shows all available wireless networks in your area. Tap the desired network to open the connection mask.

Type in the password of the wireless network (if necessary) and accept the input by pressing *Connect* to establish a connection.

The Wi-Fi network list now shows the status of the wireless network connection. *Connected* indicates an active connection.

Note: The Wi-Fi switch needs to be turned **ON** in order to see all available wireless networks.


The wireless network connection is active and can be used with the Tablet PC.

Note: If no Wi-Fi network is available, tethering via another device (e.g. smartphone) can be used to establish an internet connection.

Using the Tablet PC browser

Open the Tablet PC Browser by pressing  in the Tablet PC Launcher. The Tablet PC Browser appears.

Tablet PC Browser

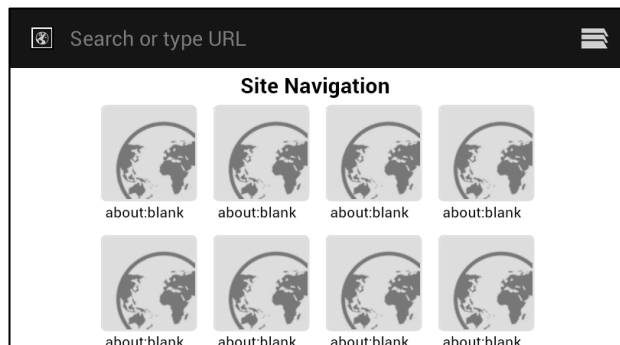

Type in the desired URL to open in the mask “*Search or type URL*”.

Press the Home-Button to return to the Launcher.

Note: An active internet connection is required to use the browser as intended.

How to update the CUBE App


If the Tablet PC already has a connection to the internet, continue with the following procedure. Otherwise, please follow the steps in chapter “Set up a wireless network connection” on page 4.

Step 1: *Update available*



If a newer version of the app is released, the text *Update available* is displayed in the lower right corner. Tap on this text.

Step 2: *Download update now*

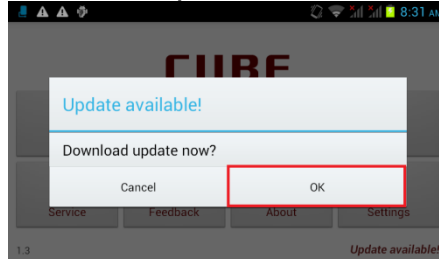


A dialog with the message *Download update now?* is displayed. Tap *OK*.

Step 3: *Downloading update*




The download starts. A toast with the text *Downloading update* is shown. Wait until the download has finished.

Step 4: *Do you want to install*

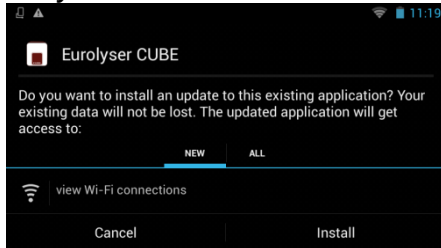


On the next screen the question “*Do you want to install this application?*” is displayed. Tap on *Install*.

Step 5: *Verify apps?*

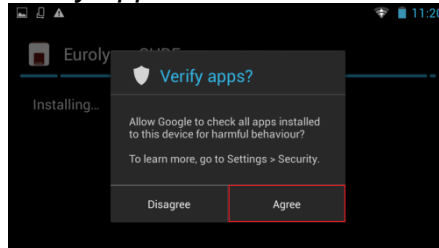


When updating the first time the “*Verify apps?*” dialog appears. Once agreeing to the dialog the installation begins. Tap on *Agree*.

Step 6: *Installing*

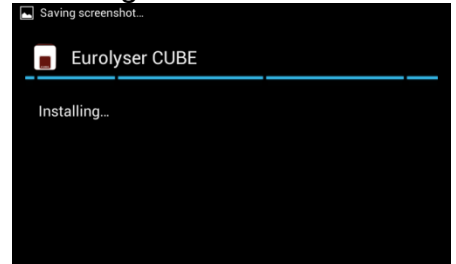

The installation begins. As long as the installation is in process this screen is displayed.

Step 7: *Installation finished*

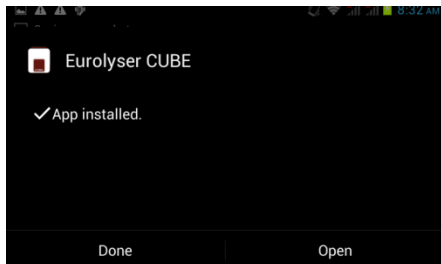

When the App is installed get back to the App by tapping *Open*.