# Troponin I Testkit

## Deutsch

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# Troponin I test kit

## English

for quantitative in vitro determination of cardiac Troponin I on smart or CUBE laboratory photometer.

Order information	Indication	Kit size
Order number: ST0230	Troponin I test kit	6 tests/pack
Order number: ST2300	Troponin I control kit	1 x 1 ml

**Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

### Summary

Cardiac Troponin I is expressed in cardiac muscle tissue by a single isoform with molecular weight 23876 Da and it consists of 209 amino acid residues. For more than 15 years cardiac Troponin I has been known in the literature as a reliable marker of cardiac muscle tissue injury and considered to be more sensitive and significantly more specific in diagnosis of the myocardial infarction than the "golden marker" of last decades – CK-MB. Published literature states that serum levels of cardiac enzymes and isoenzymes are essential to the diagnosis or exclusion of myocardial damage and that cardiac Troponin I is specific for cardiac tissue and is detected in the serum only if myocardial injury has occurred. These reports state that Troponin I determination allows early identification and stratification of patients with chest pain suggestive of ischaemia, allows identification of patients that present 48 hours to 6 days after infarction, and identifies patients with false positive elevations in CK-MB1-6.

### Method

Cardiac Troponin I Assay is based on a latex enhanced immunoturbidimetric assay. Cardiac Troponin I in the sample binds to the specific anti Troponin I antibodies, which are coated on latex particles, and causes agglutination. The degree of the turbidity caused by agglutination can be measured optically and is proportional to the amount of Troponin I in the sample.

### Measurement Range

1.00 ng/ml to 10.00 ng/ml

### Test Kit

ERS cuvette pre-filled with buffer reagent  
ERS cap pre-filled with latex reagent

### Stability and Storage

Stable until the expiration date stated on the label when stored in unopened vacuum package at 2 - 8 °C. The stability gets limited with opening the vacuum package, when stored at 2 - 8 °C, to 3 months beginning from the date of opening. The maximum stability is set by the expiration date stated on the label.

### Warnings and Precautions

This test kit is for in vitro diagnostic use only. Do NOT INGEST. Avoid contact with skin and eyes. Contains sodium azide (0.95 g/l), which may react with lead or copper plumbing to form explosive compounds. Take the necessary precautions for the use of laboratory reagents.

### Waste Management

Please refer to local legal requirements.

### Sample Material

Serum or lithium heparin plasma samples can be used for the cardiac Troponin I assay. For serum, collect whole blood by venipuncture and allow clotting. For plasma, mix the sample by gentle inversion prior to centrifugation. Centrifuge and separate serum or plasma as soon as possible after collection.

### Reference Range

Cardiac Troponin I assay was used in the reference interval study according to CLSI C28-A3 protocol with serum specimens from apparently healthy adults in the age range of 16 - 71. The reference range is recommended as < 1.50 ng/ml.

It is recommended that each laboratory establishes its own reference ranges.

### Quality Control

For internal quality control the Troponin I control kit should be used.  
Order number: ST2300

### Precision

In the study, two samples containing cardiac Troponin I were tested on smart with 20 replicates.  
Level 1; N = 20; mean 2.53 ng/ml; SD 0.163; CV 6.44%  
Level 2; N = 20; mean 6.22 ng/ml; SD 0.351; CV 5.65%

### Interferences

No interferences found for: triglyceride (500 mg/dl), ascorbic acid (10 mM), bilirubin (10 mg/dl), bilirubin conjugated (20 mg/dl), haemoglobin (200 mg/dl) and rheumatoid factor (150 IU/ml).

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### ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature before use!

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- 1. Preparation of test system**
- 1.1 Place RFID card
- 1.2 Place ERS cuvette in test kit rack
- 1.3 Place ERS cap in test kit rack
- 1.4 Press „Measurement“ button, enter required information using the touch screen

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### 2. Sample preparation serum or lithium-heparin plasma from primary tube

- 2.1 Aspirate 50 µl serum/plasma from primary tube

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### 3. Test processing

- 3.1 Dispense 50 µl sample INTO LIQUID in ERS cuvette
- 3.2 Apply ERS cap firmly onto ERS cuvette
- 3.3 Place ERS cartridge into laboratory photometer
- 3.4 Start automatic sample processing by pressing the  start button on the smart laboratory photometer, or by closing the door of the CUBE laboratory photometer.

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